

NEW


LabTEST

LABORATORY QUICK BLOOD TYPING TEST IN CATS AND DOGS


Alvedia is pleased to introduce a multiple Blood Typing Quick Test for laboratories :

LabTEST

Additional material needed :

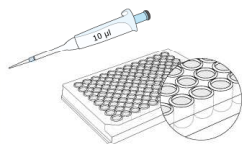

- Same monoclonal antibodies used in the Gel Test
- Very quick
- Easy to handle
- Easy interpretation
- No incubation
- No centrifugation
- Several tests can be run in the same time

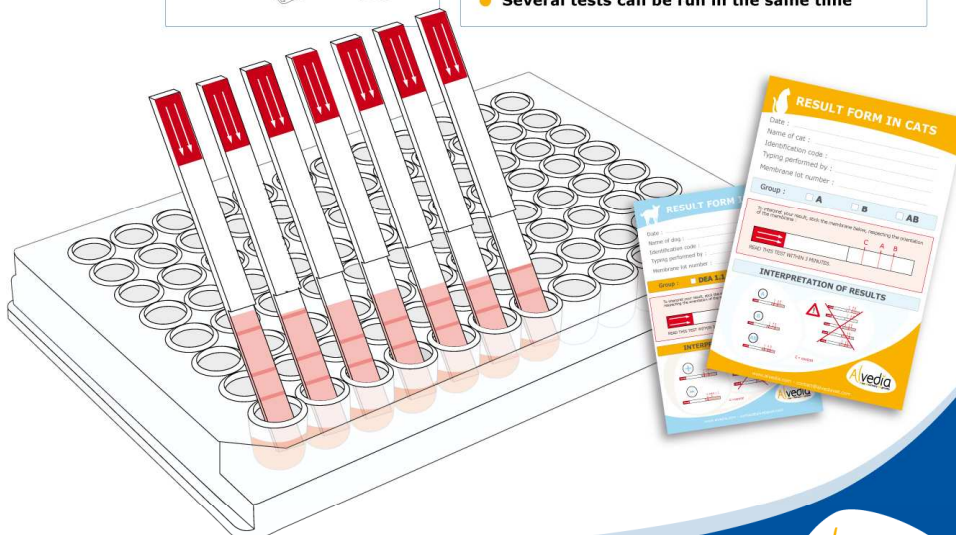

COMPONENTS OF THE LABTEST

Components	<i>lab</i> TEST _{DEA 1.1}	<i>lab</i> TEST _{A+B}
The withe tube	Contains 20 membranes (1 membrane = 1 typing) Store at Room Temperature	Contains 20 membranes (1 membrane = 1 typing) Store at Room Temperature
The buffer solution	A 5ml dropper bottle of buffer 3 drops per determination Store at 2-8°C	A 5ml dropper bottle of buffer 3 drops per determination Store at 2-8°C
Procedure time	Less than 5 minutes for the whole procedure	Less than 5 minutes for the whole procedure
The schematic user's guide	1 schematic user's guide in dogs	1 schematic user's guide in cats
The result form	This form allows you to interpret the result when fixing the membrane on the indicated space	This form allows you to interpret the result when fixing the membrane on the indicated space
The 96 well microplate (Round bottom only)	The microplate allows you to perform 96 determinations. Each well, permits to mix the 10µl of blood and the 3 drops of buffer. If you wish to purchase them from Alvedia: Price for 1 microplate: 5€	

1 BOX CONTAINS:
 20 typing membranes
 +
 1 buffer of 5ml
 +
 20 results forms

